

Homeoblock™

LATERAL EXPANSION & ENHANCED FACIAL SYMMETRY

TECHNICAL BULLETIN



Homeo Block from Johns Dental

www.johnsdental.com

The Homeo Block from Johns Dental can be used on either upper and/or lower arches.

8 BEST CASES FOR THE HOMEOBLOCK:

- 1). Relapsed Ortho
- 2). Recapturing less than 6 mm of space
- 3). Straightening teeth before veneers
- 4). Single arch treatment
- 5). Labial proclined teeth
- 6). Narrow maxillary & mandibular arches
- 7). Noticeable facial asymmetry
- 8). Aesthetic enhancement as a career choice

The Homeoblock™ from Johns Dental was designed for lateral expansion and to improve facial symmetry (or the cranio facial complex) for the adult patient. It has also has the capabilities to improve airways. Patients can expect to see some results as early as 3-4 months.

No patient compliance issues

There is a good compliance ratio with patients wearing the Homeoblock. Adults are normally motivated to achieve results and the fact that they only wear the appliance at night adds to their cooperation.

Homeoblock improves airways

Results show expansion efforts and enhanced arch development with the Homeoblock also shows improvements to the airway.

Arch expansion

The Homeoblock creates space as expansion occurs and allows the teeth to move into their new position. An additional benefit of this appliance is that it expands the arch without "tipping" of teeth.

Wearing instructions:

The Patient expands the appliance 1x per week (.2 mm of expansion). The appliance should be worn watching TV and during sleeping hours with a goal of 6-10 hours per night.

Additional interest in the Homeoblock.

Training CD's are available if you would like to become certified with the Homeoblock appliance. To become certified, contact OrthoSmile™ at (518)943-7703 or register on-line at www.facialdevelopment.com.

Johns Dental is a certified dental lab with experience in fabricating this appliance.

™OrthoSmile, Inc., Theodore Belfor, DDS
(518)943-7703
www.facialdevelopment.com

Lab Technician Quote:

"Johns Dental is a full service lab, so we can also help doctors with veneer cases that can follow Homeoblock treatment."

With the Homeoblock,™ you'll see:

- More prominent cheekbones
- Wider smile
- Decreased lines & wrinkles
- Better facial symmetry
- Improved airway



Appliances that "snap in" the first time.

423 South 13th St.
Terre Haute, Indiana 47807
Toll Free: 800/457-0504
Local: 812/232-6026
Fax: 812/234-4464
Website: www.johnsdental.com
Johns Dental Laboratories Copyright 2011

Our appliances are proudly made in the USA and only with FDA approved materials.