

ClearMet Adjustment Guidelines

Attn: Doctor

ClearMet partials can easily be adjusted with heat. We recommend using a 3 prong plier and the boiling water method to adjust the clasps on a ClearMet partial.

You can adjust the clasp by heating the pliers in boiling water or by dipping the clasps directly into the boiling water. It is important to hold the clasp in its desired shape until it is cooled. You can cool the clasp by applying cold water or with an air hose.

The clasps can also be adjusted by placing the pliers over direct heat, but our technicians have found this technique too easy to overheat the pliers causing the clasps to melt off.

Please contact the lab for any questions you may have.

